



# SD 20 ALCOHOL WIPES

## Surface Disinfectant Wipes Alcohol-based

Rev. 0 - 2020

### KEY FEATURES

For the disinfection of non invasive medical device surfaces, specially formulated for the medical and dental environment. Alcohol based formulation.

- Fresh lemon aroma
- Contact time from only 60 seconds
- Wide effective spectrum
- Kills Coronavirus
- Quick drying
- Leaves no smears or residue
- Two sizes available

SD20 are ideal to use on alcohol resistant, wipeable surfaces within the dental surgery where short contact times are important, eg in between patients. The highly effective solution is Bactericidal, Yeasticidal, Tuberculocidal, Mycobactericidal and Virucidal against enveloped viruses including the deadly Coronavirus, HIV & Hepatitis (see table opposite for full details). The wipes measuring 180x120mm and packs containing 100 wipes with a dispenser tubs.

### COMPOSITION

70.5% v/v Ethanol, Aqua, Parfum, Linalool, Limonene.

### INSTRUCTIONS FOR USE:

Using a new wipe for each application wipe objects to be disinfected so that they are completely moistened and let the disinfectant take action for a minimum of 60 seconds (to achieve full efficacy leave for 5 minutes). Allow surface moisture to evaporate. Do not use on surfaces sensitive to alcohol. For professional use only.

### PACKAGING

**SD20**  
Code 070520.... (12) bags (100 wipes 180x120 mm)  
(1) empty tube

All primary packaging are compliant to the technical specifications provided by the Pharmacopoeia. The material used does not contain latex and it is perfectly compatible with all the components of the formulation.

Medical Device Class IIb  
CE n° 1639



### INFORMATION FOR THE EXCLUSIVE USE OF HEALTHCARE PROFESSIONALS

Manufactured by:  
Topdental (Products) Ltd.  
12 Ryefield Way, Silsden, West  
Yorkshire. BD20 0EF. England UK.  
Tel +44 (0)1535 652 750 [www.topdental.co.uk](http://www.topdental.co.uk)



Distributed exclusively by:  
**IDS S.p.A.**  
Via S. Cristoforo 28/10  
17100 Savona - Italy  
[www.ids dental.it](http://www.ids dental.it) - [info@idsdental.it](mailto:info@idsdental.it)

# SD 20


## ALCOHOL WIPES

Surface Disinfectant Wipes  
Alcohol-based

### MICROBIOLOGICAL EFFICACY

MICRO-ORGANISM	TIME
<b>Bactericidal</b>	
Pseudomonas aeruginosa	1 min
Staphylococcus aureus	1 min
Enterococcus hirae	1 min
E. Coli	1 min
VRE	1 min
MRSA	1 min
<b>Yeasticidal</b>	
Candida albicans	1 min
<b>Enveloped Viruses</b>	
Hepatitis B	1 min
Hepatitis C	1 min
HIV	1 min
Human T Cell Leukemia Virus	1 min
Measles Virus	1 min
Hepatitis Delta virus (HDV)	1 min
Influenza Virus	1 min
Coronavirus	1 min
Herpesviridae	1 min
Filoviridae (Ebola)	1 min
Rubella Virus	1 min
Rabies Virus	1 min
Flavivirus	1 min
Poxviridae	1 min
<b>Non-Enveloped Viruses</b>	
Adenovirus 5	1 min
Norovirus	1 min
<b>Mycobactericidal</b>	
Mycobacterium avium	1 min
<b>Tuberculocidal</b>	
Mycobacterium terrae	1 min
<b>Fungicidal</b>	
Aspergillus brasiliensis	5 mins

### PATHOGENIC AGENTS CLASSIFIED ACCORDING TO THEIR RESISTANCE TO DISINFECTANTS

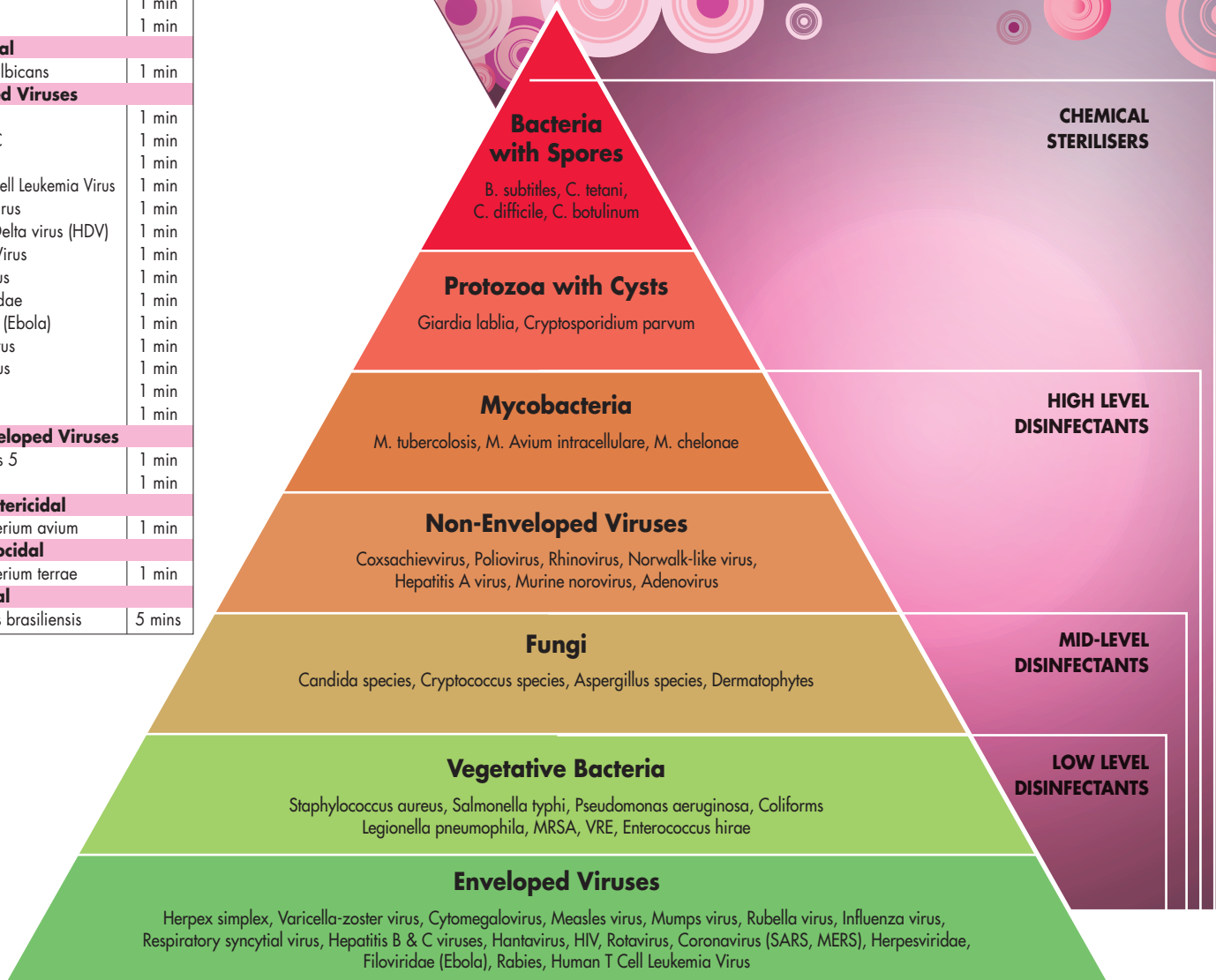

### PHYSICAL PROPERTIES

Colourless liquid with an alcohol/lemon odour  
Biodegradable  
pH: 6.5 - 8.5  
36 month shelf life from date of manufacture  
Textured substrate white in colour

### TEST STANDARDS

The following test standards have been achieved:  
EN 14885, EN 14476, EN 14348, EN 13727,  
EN 13624.

### INFORMATION FOR THE EXCLUSIVE USE OF HEALTHCARE PROFESSIONALS