virooxid® ASPIRATOR

ASPIRATOR CLEANER AND DISINFECTANT CONCENTRATE

GB Aspirator Cleaner and Disinfectant Concentrate

For the daily cleaning and maintenance of Dental Aspirator Systems. Can be used on wet line, dry line and semi wet/dry line suction systems

Instructions for use

Daily: Add 20ml of concentrate to 980ml warm water (2% solution).

Aspirate via the high and low volume suction tubes. Leave for a minimum of 15 minutes (to achieve full efficacy leave for 60 minutes) and then rinse thoroughly with clean water. Do not use on aluminium or copper parts. For professional use only.

Warnings

Danger (Applies to concentrate, not diluted solution)

H314: Causes severe skin burns and eye damage.

P260: Do not breathe dust/fumes/gas/mist/vapours/spray.

P280: Wear protective gloves/protective clothing/eye protection/face protection.

P301+330+331: IF SWALLOWED: rinse mouth. Do NOT induce vomiting.

P303+361+353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.

P304+340: IF INHALED: Remove person to fresh air and keep comfortable for breathing.

P305+351+338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **Storage.** P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool.

Contra indications. Incompatible materials: Strong acids. Strong oxidising agents.

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TOP DENTAL Products Ltd