True Light

Lamp
for dental photography
with smartphone

Instruction Manual	EN	2
Manuale d'istruzioni	IT	4
Gebrauchsanleitung	DE	6
Manuel d'instructions	FR	8
Manual de instrucciones	ES	10
Manual de instruções	PT	12
Gebruikshandleiding	NL	14
Brugsanvisning	DA	16
Käyttöopas	FI	18
Bruksanvisning	NO	20
Användarmanual	SV	22
Instrukcja obsługi	PL	24
Návod	CZ	26
Návod	SK	28
Manual de instrucțiuni	RO	30
Руководство по эксплуатации	RU	32
Talimat El kitabı	TR	34
دليل التعليمات	AR	36

True Light

CE   FC RoHS

www.blancone.eu

True Light

**Lamp for dental photography
with smartphone**

BlancOne® CARE True Light

Lamp for dental photography with smartphone

CONTENTS

- TrueLight: Lamp with support for smartphone
- Type-C charging cable
- Hex key to adjust smartphone clamp (6,7)
- Aluminum tripod
- Adjustable support
- Wireless Bluetooth remote control
- Lamp case and accessories
- Instructions


Congratulations on purchasing your TrueLight lamp. Before using it for the first time, you are advised to carefully read and follow the instructions shown below.

WARNINGS

TrueLight is a high-tech device that must be used with caution. Any problems due to incorrect use are not covered by the warranty.

Practitioners must warn patients not to look directly at the light source (LED lighting), and to wear protective glasses if necessary.

PRODUCT DESCRIPTION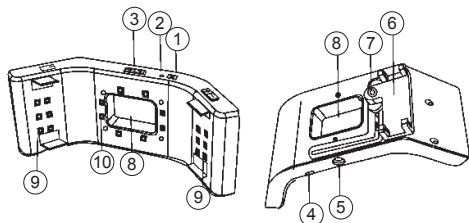

FIRST TIME USE

Use the cable supplied in the package to charge the TrueLight battery. Insert the Type-C cable into the socket beneath the TrueLight (4) and plug the USB end into the charger, connecting it to the power supply.


The red light (2) above the device will turn on while it is charging, the yellow light will indicate that there is insufficient power, and the green light will come on when it is fully charged.

TRUE LIGHT takes approximately 5 hours to fully recharge. Once fully charged, the battery has approximately 18 hours of autonomy.

CLAMP FOR SMARTPHONE


The smartphone clamp (6) on the back of the TrueLight is adjustable. The smartphone clamp can hold devices with a width of 55-85mm, making it suitable for most smartphones on the market.

The hex screw (7) on the mobile carriage of the clamp can be loosened in order to slide the carriage and adjust the position and height of the smartphone, so that the lens remains in the center of the TrueLight window (8).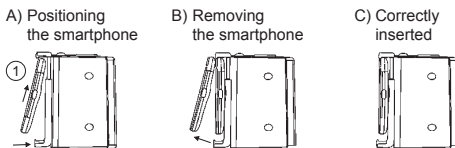

Without using excessive force, follow these instructions to position the clamp:

1. Insert the smartphone into the clamp as shown below (A). Press the smartphone on the upper support to open the clamp and insert the device.
2. Slide the smartphone into the clamp to correct the position of the lens.
3. After positioning the smartphone, you may need to move the clamp to position the target in the center of the window.
 - a. Gently push the smartphone through the spring of the slot to remove it (B)
 - b. Use the millimeter scale on the back of the TrueLight to facilitate adjustment.
 - c. Loosen the hex screw (do not remove it completely) and gently slide the carriage of the clamp in the required direction.
 - d. Tighten the hex screw.
4. Make sure that the lens of your phone is centered in the TrueLight window. If not, repeat step 3.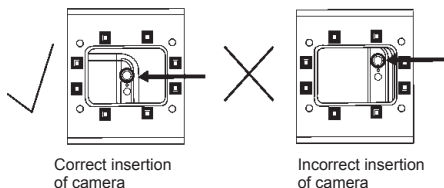

Correct insertion of camera

Incorrect insertion of camera

OPERATING THE TRUE LIGHT

The main On/Off switch (1) is on the top of the TrueLight.

TrueLight is equipped with LED lights arranged as follows:

- A central group of eight lights (10) around the window
- Two groups of six lateral LED lights (9)

A series of +/- buttons at the top of the TrueLight (3) to adjust the intensity of the light gradient separately for each of the three groups.


To dim or switch off a group of LED lights, press the corresponding(-) button. Similarly, press the (+) button to increase the intensity or switch on a group of LED lights. There are four levels of intensity.

TrueLight maintains the intensity selected when switched off and on again.

OPTIONAL ACCESSORIES

- Diffuser - mounted on the left and right LED blocks (2 pieces)
- Filter - mounted on the 8 central LEDs (1 piece)

The diffusers and filter must be securely installed on the front of the TrueLight.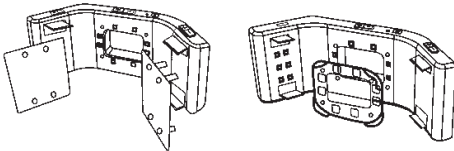

Note: The sunken nut (5) in the lower part of the True Light (next to the Type-c port), is used to secure the lamp to the tripod or the adjustable support if necessary.

TIPS FOR DENTAL PHOTOGRAPHY

FRONT TEETH

- Use the automatic mode in the basic camera application of your smartphone.
- Switch on the LED lights on the left and right side using the light diffusers if necessary.
- Maintain a distance of 15-20 cm from the patient.
- As the lens moves towards/away from the teeth, the color on the screen will change as the smartphone adjusts to the changing light. Take the picture when the screen shows the color that is closest to what you can see with the naked eye.
- Try to enlarge or reduce the image on your smartphone screen to select the optimum size of image.

- Position the device so that the smile line is parallel to the horizontal margins of the photograph.
- Position the AF (Auto Focus) point by touching the phone screen to make the image clearer.
- The shooting can be controlled by the Bluetooth remote control connected to the smartphone.

BACK TEETH


- Use retractors and mirrors if necessary.
- Switch on the central LED light.
- Maintain a distance of 15-20 cm from the patient.
- Try to enlarge or reduce the image on your smartphone screen to select the optimum size of image.
- Position the AF (Auto Focus) point by touching the phone screen to make the image clearer.
- The shooting can be controlled by the Bluetooth remote control connected to the smartphone.

INSTRUCTIONS FOR THE BLUETOOTH REMOTE CONTROL

When the on-off switch is pressed, the AB Shutter3 remote will go into pairing mode and the LED indicator will flash rapidly.

Available devices can be detected by activating the Bluetooth function on your smartphone. Select the "AB Shutter3" device from the list and the device will automatically connect in seconds.

Press the large button A if taking photographs using iOS smartphones, or the small button B for Android devices.

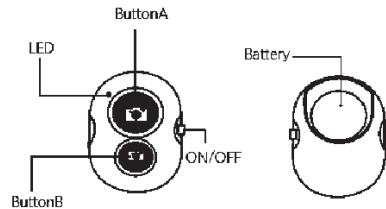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