1. Description of device

1.1 Description

Ultrasonic scaler handpiece is the accessory of scalars. It is the handheld energy transformation part of ultrasonic piezo scaler, by using piezoceramics to transfer electric energy to mechanical energy to eliminate the dental calculus.

1.2 Model and Specifications

- 1.2.1 Model: (See package label for details)
- 1.2.2 Specification: Detachable handpiece with LED light

1.3 Structure and composition:

The handpiece is composed of cover and ultrasonic transducer.

1.4 Scope of application

The handpiece is used for the dental calculus elimination and root canal cleaning.

1.5 Contraindication

- 1.5.1 The hemophilia disease patient is not allowed to use this equipment.
- 1.5.2 The patients with heart pacemaker are forbidden to use this equipment.
- 1.5.3 The heart disease patient, pregnant woman and children should be cautious to use the equipment.

1.6 The classification of the device

- 1.6.1 Operating mode: Continuous operation
- 1.6.2 Protection against electric shock: Class II equipment
- 1.6.3 Protection degree against water(used on the pedal): IPX1
- 1.6.4 Degree of safety of application in the presence of a Flammable Anaesthetic Mixture with air or with Oxygen or Nitrous Oxide : Equipment not suitable for being used in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide

- 1.7 The main technical specification
- 1.7.1 Power input: 190VAC ~ 230VAC
- 1.7.2 Output primary tip Vibration excursion: 1µm ~ 200µm
- 1.7.3 Output tip Vibration frequency: 28kHz ~ 42kHz
- 1.7.4 Output half-excursion force: 0.1N ~ 2N
- 1.7.5 Tip's output power: 3W ~ 20W
- 1.7.6 Water pressure: 0.01 MPa ~ 0.5 MPa


1.8 Working Condition

- 1.8.1 Environmental temperature: +5° C ~ +40° C
- 1.8.2 Relative humidity: 30% ~ 75%
- 1.8.3 Atmospheric pressure: 70kPa ~ 106kPa
- 1.8.4 Inflow water temperature: ≤ +25° C

2. Installation of product

2.1 Installation of handpiece

- 2.1.1 Open the packing box, make sure that all the parts and accessories are complete according to the packing list. Contact with dealer if necessary.
- 2.1.2 Keep dry the handpiece connector and cable plug socket before installing the handpiece.
- 2.1.3 Make sure that the bottom jack point is at the plug, make the handpiece fixed to the cable correctly. (See picture 1)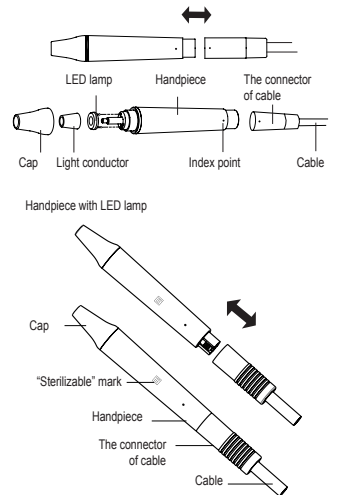


Picture 1

2.2 Installation of tips

- 2.2.1 Fix the tip to handpiece by torqueWrench until hearing the "Ka Ka" sound.
- 2.2.2 Diagram of the scaler tips, file holder installation and connection. (See picture 2)

Notes: The detachable handpiece is a high-tech device which should be used carefully. In case stepping on the foot switch by accident, unscrew the scaling tips after each operation. **Warning:** Equipment not suitable for being used in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide

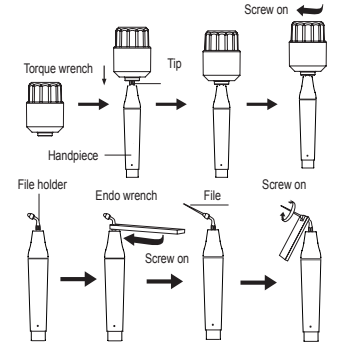Picture 2

3 Product function and operation

3.1 Scaling function

- 3.1.1 Connect the handpiece and the connector correctly, and adjust the water and vibration intensity to a suitable level.
- 3.1.2 The handpiece can be handled in the same gesture as a pen in hand.
- 3.1.3 Select a suitable scaling tip as you need, screw it on the handpiece tightly by the torque wrench.
- 3.1.4 The machine works at a very fast frequency during normal operation. With the normal vibration of the working tips and the normal atomisation of the water, you only need to touch the teeth gently and move them back and forth at a certain speed to remove the calculus, and there is no obvious feeling of heat in the working tips.
- 3.1.5 Don't make the tip overexert on the surface of the teeth in case of hurting the teeth and damaging the tip.
- 3.1.6 After finishing operation, keep the machine working for 30 seconds on the water supply condition in order to clean the handpiece and the scaling tip.

3.2 Instructions for the use of the main components

- 3.2.1 Cap: It can be screwed out so that the user can remove the cap regularly for cleaning.
- 3.2.2 Handpiece: main part, High temperature sterilizable.
- 3.2.3 "O" ring: The "O" ring is watertight and should be replaced in the event of breakage or excessive wear.
- 3.2.4 LED lamp: illuminate the oral area, detachable and sterilizable.

4. Notes

- 4.1 Don't pull out the handpiece when in operation.
- 4.2 Pull out the handpiece and remove the tip after operation.
- 4.3 Please use the original factory fitted tips. The internal screw thread of the scaling tips produced by some manufactures is coarse, rusty and collapsed. This will damage the external screw thread of the handpiece irretrievably. Please use Manufacturer brand scaling tips.
- 4.4 We are a company specialising in the manufacture of medical devices and are only responsible for the safety of these products if maintenance, repairs and alterations are carried out by us or our authorised distributors and if the replacement parts are original to us and are operated in accordance with the instructions for use.

5. Trouble shooting

Fault	Possible causes	Solutions
Handpiece doesn't work	The tip is loosened	Tighten it
	Handpiece fault	Take out the handpiece and send it back for repair
	Cable fault	Contact our dealers or us
There is no spray when getting through the electricity	The connect plug between the cable and circuit board is loosened	Contact our dealers or us
	The water control switch is not on	Turn on the water control switch
There is no water when getting through the electricity	No pressure or low pressure	Check the pressure
	The water control switch is not on	Turn on the water control switch
Low vibration	There is impurity in the solenoid valve	Contact our dealers or us
	Water way clogs	Clean the water line by three-way syringe
	The tip hasn't been screwed on the handpiece tightly	Screw the tips tightly by torque wrench
The handpiece generates heat	The tip is shaken loose by vibration	Screw on the tip tightly
	The coupling between the handpiece and the cable isn't dry	Dry it, especially the water between handpiece and connector
	The tip is worn out or deformed	Change another tip
There is water seeping from the coupling between the handpiece and the cable	The waterproof "O" ring is damaged	Turn the water control switch to a higher grade
	Poor contact	Contact tightly
LED light doesn't work	Something wrong with LED light	Change a new one
	LED lamp installed backwards	Please install the "+" of the LED lamp to the "+" of the handpiece

If the problem still can't be solved, please contact with local dealer or manufacturer.

6. Cleaning, disinfection and sterilization

- 6.1 Clean and sterilize the handpiece after each use.
- 6.2 Remove the cap, light pipe, and LED lamp before autoclaved, clean by pure water and dry it by three-way syringe.
- 6.3 The recommended sterilization procedure for autoclaves is: sterilization temperature 134° C, sterilization time 4 minutes, vacuum 3 times, drying time 15 minutes.
- 6.4 Wrap the handpiece in a disinfectant towel or bag and then disinfect it. Allow the handpiece to cool naturally before attaching the LED light, the light guide and the cap to the handpiece in the order shown in picture 1 before using it again to avoid burns.
- 6.5 Cleaning and disinfection precautions
 - 6.5.1 Clear the cleaning liquid in the handpiece with three-way syringe before sterilization.
 - 6.5.2 Be sure that the scaling tip has been unscrewed from the handpiece and it cannot be sterilized with others.
 - 6.5.3 Please confirm whether the outer of the handpiece is damaged during the treatment or sterilization.
 - 6.5.4 Please don't smear any protective oil on the surface of handpiece.
 - 6.5.5 Please take out the handpiece, nipple, light pipe, LED lamp as Picture 3 and sterilize separately.
 - 6.5.6 The water connector of the cable connector has two waterproof O-rings. To prolong their service life, they should be lubricated regularly with dental lubricant as they need to be inserted and removed repeatedly. In case of breakage or excessive wear, the watertight O-rings should be replaced in good time. The watertight O-rings should be replaced as soon as they are damaged or excessively worn.
- 6.6 The following sterilizing methods are forbidden:
 - 6.6.1 Put handpiece into any liquid for boiling.
 - 6.6.2 Dip handpiece in disinfectors such as iodine, alcohol and glutaraldehyde.
 - 6.6.3 Put handpiece into oven or microwave oven for baking.
 - 6.6.4 Put handpiece into ultrasonic cleaner for cleaning.

7. Transportation, storage and maintenance

- 7.1 Transportation
 - 7.1.1 Excessive impact and shake should be prevented during transportation. Lay it carefully and lightly and don't invert it.
 - 7.1.2 Don't put it together with dangerous goods during transportation.
- 7.2 Storage
 - 7.2.1 Don't store the machine together with the articles which are combustible, poisonous, caustic, or explosive.
 - 7.2.2 This equipment should be stored in a room where the relative humidity is 10% -93%, atmospheric pressure is 70kPa -106kPa, and the temperature is -20° C ~ +55° C.
- 7.3 Maintenance
 - The equipment should be handled carefully and lightly. Be sure that it is far from the vibration, and installed or kept in a cool, dry and ventilated place.

8. Environmental protection

There is not any harm factor in our products. You can deal with it based on the local law.

9. After service

If the equipment does not work properly due to quality problems within one year from the date of sale, we will be responsible for repairing it free of charge with the warranty card, for details refer to the warranty description in the warranty card.

10. Symbol instruction


	Caution
	Refer to instruction manual/booklet
	Date of manufacture
	Manufacturer
	Class II equipment
	Can resist water that drips vertically onto the product.
	For indoor use only
	Waste electrical and electronic equipment
	Atmospheric pressure limitation: 70kPa-106kPa
	Temperature limit: -20° ~ +55°
	Humidity limitation: 10% ~ 93%

11. Statements

All rights of modifying the product are reserved to the Manufacturer without further notice. The pictures are only for reference. The final interpretation rights belong to the Manufacturer. The industrial design, inner structure, etc., have been claimed for several patents by manufacturer, any copy or fake product must take legal responsibilities.

Before installing the handpiece, make sure that the handpiece connector and the cable plug socket are completely dry.

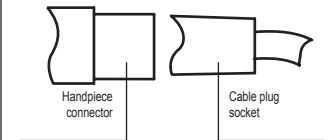IDS Spa
17100 Savona - Italy
Via Valletta San Cristoforo, 28/10
Tel. +39 019 862080
www.ids dental.it

Guilin Refine Medical Instrument Co., Ltd.
No.8-3, Information Industrial Park, High-Tech Zone, Qixing District, 541004 Guilin, Guangxi, PEOPLE'S REPUBLIC OF CHINA

EC REP MedNet EC-REP C Iib GmbH
Borkstrasse 10, 48163 Münster, Germany

UK RP MediMap Ltd
2 The Drift Thurston Suffolk IP31 3RT
United Kingdom

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IDS SpA
EC REP MedNet EC-REP C Ilb GmbH