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EN OPTICORE DUAL BOND ADHESIVE Dual Cure Bonding for the Total Etch Technique

Product description

OPTICORE DUAL BOND is a simple to use dual cure bonding consisting of the light cure component OPTICORE DUAL BOND ADHESIVE that after mixing with OPTICORE DUAL BOND ACTIVATOR gives a dual cure bonding. It is designed for strong bonding of light cure, self cure and dual cure composites to etched enamel and dentine and to not precious and precious metals. OPTICORE DUAL BOND can be used for priming the root canals before cementation of endodontic posts with self- or dual cure composites (e.g. OPTICORE). The strong adhesion of OPTICORE DUAL BOND to etched enamel or dentine proceeds on principles similar to that occurring with glass ionomer cements. Good, long lasting adhesive strength and good biocompatibility are attained by polycarboxylic acids.

OPTICORE DUAL BOND can be used on slightly moist dentine surfaces (wet bonding technique).

Indications/Intended use

Adhesive for:
• Restorations with light cure, self cure and dual cure composites

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of OPTICORE DUAL BOND. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.

APPLICATION

1. Application of OPTICORE DUAL BOND in Root Canals

1.1. Isolation

Use of a rubber dam to isolate the tooth is strongly recommended.

1.2. Root Canal Preparation

Prepare and clean the root canal according to the instructions of the selected post manufacturer. Dry the root canal but do not desiccate. Etch the root canal with OPTIETCH EXTREME for 15 seconds and rinse with water using an endodontic irrigation syringe. Blot the canal dry with soft paper tips, leaving the dentin visibly moist. A slightly wet dentine surface is important for the function of OPTICORE DUAL BOND.

Etching Precaution:

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

1.3. Application of OPTICORE DUAL BOND

One drop of OPTICORE DUAL BOND ADHESIVE and one drop of OPTICORE DUAL BOND ACTIVATOR were combined in a mixing pallet and mixed for 5-10 seconds under subdued light.

Note:

Do not interchange lids of the bottles, because this can lead to a cross-contamination of the liquids.

Apply OPTICORE DUAL BOND generously with a syringe or a suitable brush on the prepared root canal walls for 30 seconds with agitation. The material should build a homogeneous layer.

Remove excess material carefully e.g. with paper points. Dry cautiously for about 15 seconds with oil free air to remove all volatile components and to disperse the adhesive to an even layer. Do not desiccate.

Optional cure the OPTICORE DUAL BOND coating by exposing its entire area to a dental halogen light unit for 20 seconds before application of a second layer of OPTICORE DUAL BOND.

1.4. Application of a 2nd Layer of OPTICORE DUAL BOND

Apply again the OPTICORE DUAL BOND generously with a brush onto the adhesive surfaces as described under 1.3. Optional light cure. It is essential that the primed dentine and enamel surfaces are dry and contaminant free for the application of the composite. Proceed immediately with placement of the post.

1.5. Post Cementation

Prepare the post according to manufacturer instructions.

Place the post with a flowable dual cure or self cure composite (e.g. OPTICORE) into the root canal. For post stabilization light cure the coronal part of the cemented post for 20 seconds.

2. Composite Fillings and Indirect Restorations

2.1. Isolation

Rubber dam is the recommended method of isolation.

2.2. Cavity Preparation

Clean the tooth prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (0.5 - 1.0 mm) bevel placed in the enamel to increase the surface area for greater bond strength.

2.3. Pulp Protection

For pulp protection areas close to the pulp should be covered with a thin layer of calcium hydroxide material.

2.4. Enamel and Dentine Conditioning

Apply OPTIETCH EXTREM onto the enamel and dentine surfaces beginning with the enamel bevels.

Condition the enamel for at least 15 seconds and the dentine for 15 seconds. (This results in 20-30 seconds etching of enamel and 15 seconds of dentine). Deciduous teeth are etched correspondingly longer.

Rinse for 20 seconds with water. Dry it in a water and oil-free airstream, but do not desiccate. A slightly wet dentine surface is important for the function of OPTICORE DUAL BOND. The etched enamel bevel should have a chalky white appearance.

Etching precaution:

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

2.5. Application of OPTICORE DUAL BOND

For light cure composites:

Apply OPTICORE DUAL BOND ADHESIVE generously with a brush onto the enamel and dentin surfaces for 30 seconds with agitation. The material should build a homogeneous layer. Remove excess material carefully. Dry cautiously with oil free air for about 15 seconds to remove all volatile components and to disperse the adhesive to an even layer. Do not desiccate the dentine.

Cure the OPTICORE DUAL BOND ADHESIVE coating by exposing its entire area to a dental halogen light unit for 20 seconds before application of a second layer of OPTICORE DUAL BOND ADHESIVE.

Notes:

Do not rinse off the OPTICORE DUAL BOND ADHESIVE! If not used immediately, place dispensed OPTICORE DUAL BOND ADHESIVE in subdued light to prevent premature polymerization by incident light. The OPTICORE DUAL BOND ADHESIVE will not self cure.

Apply again the OPTICORE DUAL BOND ADHESIVE generously with a brush onto the adhesive surfaces as described above. before placement of a light cure composite.

For self cure and dual cure composites:

One drop of OPTICORE DUAL BOND ADHESIVE and one drop of OPTICORE DUAL BOND ACTIVATOR were combined in a mixing pallet and mixed for 5-10 seconds under subdued light.

Note:

Do not interchange lids of the bottles, because this can lead to a cross-contamination of the liquids.

Apply the OPTICORE DUAL BOND mixture generously with a brush onto the enamel and dentine surfaces for 30 seconds with agitation. The material should build a homogeneous layer. Remove excess material carefully. Dry cautiously with oil free air for about 15 seconds to remove all volatile components and to disperse the adhesive to an even layer. Do not desiccate the dentine.

Cure the OPTICORE DUAL BOND coating by exposing its entire area to a dental halogen light unit for 20 seconds before application of a second layer of OPTICORE DUAL BOND. Without light cure apply the second layer after drying with oil free air.

Apply again the OPTICORE DUAL BOND mixture generously with a brush onto the adhesive surfaces as described above before placement of a composite.

It is essential that the primed dentine and enamel surfaces are dry and contaminant free for the application of the composite.

The self cure or dual cure composite can be applied immediately.

2.6. Application of the Composite

2.6.1. Direct Restorations

Apply the restorative material according to the instructions of the manufacturer.

Light cure restorative materials:

Best results are obtained with application of a thin layer of a light cure flowable composite (e.g. OPTIFILfil) followed by the application of a moldable composite (e.g. OPTIFIL). Light cure each composite layer separately according to the corresponding user instructions.

Self or dual cure restorative materials:

Apply a self cure or dual cure composite according to the user instructions. Light cure in case of dual cure composites shortens setting time.

2.6.2. Indirect Restorations

Refer to manufacturers instructions for application of indirect restorations.

For cementing of the indirect restoration a self cure or dual cure composite cement is applied according to the user instructions.