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GB) INSTRUCTION FOR USE:

Core build-ups and cementation of posts with Opticore System

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1) PREPARATION

adjust the post cavity depth (approximately 5-7 mm) with corresponding precision drill.

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2) ETCHING

Etch dentine surfaces with phosphoric acid for 15 seconds. Wash and dry

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3) DUAL BONDING

- place 2 layers of Opticore Dual Bond mixed at 1:1 ratio with catalyst. Dry with oil free air.

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4) CEMENTATION

Place OPTICORE directly into the cavity using the Oral Tip with Endo Tip.

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5) ENDODONTIC POST PLACEMENT

Insert carbon or glass post applying enough strength. Light cure for 20 seconds in order to hold the post in position.

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6) CORE BUILD-UP

Build up core with Opticore. Light cure for 40 seconds or leave self-curing for 3:30 minutes.

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COMPARISON TABLE: This table shows that OPTICORE is much more flexible compared to competitor products, but similar hardness and compressive strength, thus allowing a better overall strength to bite forces into the mouth. Also, thanks to its lower opacity, ensures a better esthetic performance under metal free crowns.

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EN OPTICORE Classic Dual Curing Flowable Hybride Composite for Core Build-ups and Cementation of Posts

Product description

OPTICORE is a **dual curing radiopaque flowable hybride composite** based material for core build-up and cementation of posts.

OPTICORE is based on poly- and difunctional methacrylates and inorganic filler particles of 0.05-2.5 µm. The total filler content is 66 % by weight and 46 % by volume.

Delivered in auto-mixing 1:1 cartridges it can be easily dispensed with an application gun and applied directly. OPTICORE exhibits a short setting time without high heat generation. The dual cure properties enables the dentist, also to carry out cementations and core build ups in cases where a light cure cannot be guaranteed to be sufficient. OPTICORE meets the requirements of the international standard **ISO 4049**, type 2, class 3.

Indications/intended use

- Core build-ups
- Cementing of posts
- Cementing of crowns and bridges, inlays and onlays

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

The placement of OPTICORE Classic is contraindicated if a dry working area or the recommended application technique are not possible. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

1. Cavity Preparation

Use of a rubber dam to isolate the tooth is strongly recommended. Remove all existing old restorations and decay from the tooth. If necessary place any pins or posts. Refer to directions of the selected post manufacturer.

2. Pulp Protection

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calcium hydroxide material.

3. Etching

Etch the enamel and dentine according to the

instructions of the product.

4. Application of OPTICORE DUAL BOND

Apply OPTICORE DUAL BOND according to the user instructions.

5. Preparing of the cartridge

Take off the turn cap of the double cartridge and throw it away (do not use it again!). It is substituted by the special 1:1 mixing cannula by attaching it with a sideways turning movement, locking the cannula in place.

The application gun is easy to load with the prepared cartridge and is ready for application.

Note: The initial extrusion from the mixing cannula (about the size of a pea) should be discarded for each new mix.

Store used cartridge with fixed used mixing cannula in the dark.

The working time (23°C (74°F)) of OPTICORE in the self cure mode is 1:30 minutes from start of mixing.

6. Application

Place the mixing cannula directly into the preparation and press out the paste.

OPTICORE is automatically mixed when dispensed with slight and even pressure. Filling should occur from bottom upward to prevent voids. To facilitate placement of OPTICORE place a matrix band around the prepared tooth.

OPTICORE may be contoured by using a composite instrument. A flat ended interproximal carver is recommended.

Place OPTICORE directly into the preparation and allow the system to self cure for 3:30 minutes. After that the material should be light-cured (40 seconds) with a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm². With this technique an optimum of physical properties will be obtained. An explorer can be used to test that the OPTICORE is completely set. Remove the matrix net earlier than the material has set.

Final core preparation on the OPTICORE can be carried out by using crown preparation burs.

7. Post Cementation

Prepare the selected post according to manufacturer directions. Before cementing the post prepare the root canal in the usual manner, etch and apply a suitable bonding agent.

OPTICORE is applied into the prepared root canal and onto the post. Seat the post careful into the canal and maintain firm pressure until the post is seated. OPTICORE self-cures within **3:30 minutes**. For post stabilization light cure the coronal part of the cemented post for **20 seconds** with a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm².

Additional Notes/Warnings

The ambient light of the dental lamp may start polymerization of the composite. Do not use any resin to adjust viscosity of the composite.

Avoid contact with skin, mucous membrane and eyes.

If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.

Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates. Color stability meets the requirements of DIN EN ISO 4049.

Commercial medical gloves do not protect against the sensitizing effect of methacrylates. Keep away from children!

Composition

Dimethacrylates, dental glass, silicone dioxide, photo initiators, catalysts

Storage

Do not store above 20 °C (68 °F). Protect from direct sunlight. Store unopened material in the refrigerator. Opened cartridges have to be used up within 3 months. Do not use after expiry date.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

SanaPro Dental GmbH warrants this product will be free from defects in material and manufacture. SanaPro Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and SanaPro Dental GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, SanaPro Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!