I

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CE 0546 Rev3 - Sept 16, 2014

The logo for International Dental Supply (IDS) features the letters 'IDS' in a bold, stylized font. The 'I' and 'D' are connected, and the 'S' is separate. The letters are white with a grey shadow effect, set against a dark background.

INTERNATIONAL DENTAL SUPPLY

International Dental Supply Srl
Via Valletta San Cristoforo 28/10
17100 Savona (SV) - ITALY

Tel +39 019-862080

www.idsdental.it

email: info@idsdental.it

OPTICORE POST Endodontic Precision Drills

Opticore Post endodontic precision drills, are available for all kind and size of Opticore fiber posts.

Please follow Opticore Post user's instructions as concern the use of drills during endodontic canal preparation.

NOTE: Endodontic drills can be used to determine the usefull length of the post.

INSTRUMENT CARE

After each use, the instrument must be thoroughly cleaned of debris and rinsed thoroughly under running water. Must be moist heat sterilized at 134° C for 5 minutes complete cycle. 10 minutes dry time. With regard to sterilization mode, refer to the instructions for use supplied by the manufacturer of the sterilizer. In any case you must before sterilization operations, clean the device with water and place it inside the special disposable bags before placing the device inside the autoclave.

WARNING: it is the responsibility of the user to pack the device properly before sterilization.

Using soda, salt, mercury as well as chemiclaving, could compromise shining and aspect of the instrument without reducing its cutting efficiency.

Can be used up to a decrease in cutting capacity.

CE 0546 Rev3 - Sept 16, 2014

INTERNATIONAL DENTAL SUPPLY

International Dental Supply Srl
Via Valletta San Cristoforo 28/10
17100 Savona (SV) - ITALY
Tel +39 019-862080
www.ids dental.it
email: info@idsdental.it