OPTIFIL BOND

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Product description

OPTIFIL BOND is a simple to use **light cure one component bonding agent**. It is designed for strong bonding of light cure composites to etched enamel and dentine.

Its strong adhesion to etched enamel or dentin proceeds on principles similar to that occurring with glassionomer cements. Good, long lasting adhesive strength and good biocompatibility are attained by polycarboxylic acids.

OPTIFIL BOND is compatible with all current brands of visible light cure composite materials. It is ethanol-based and hydrophilic. OPTIFIL BOND can be used on slightly moist dentin surfaces (wet bonding technique).

Indications/Intended use

Adhesive for:

- Restorations with light cure composites

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol

inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (**0.5 - 1.0 mm**) bevel placed in the enamel to increase the surface area for greater bond strength.

3. Pulp Protection

Cavity floor of deep excavations should be covered with a thin layer of calcium hydroxide material.

4. Enamel and Dentine Conditioning

Recommended is the total etch technique. Apply **OPTIETCH EXTREME** onto the enamel and dentine surfaces beginning with the enamel bevels. Condition the enamel for at least **15 seconds** and the dentine for **15 seconds**. (This results in 20-30 seconds etching of enamel and 15 seconds of dentine). Deciduous teeth are etched correspondingly longer. Rinse for 20-seconds with water. Dry it in a water and oil-free airstream, but **do not** desiccate. A slightly wet dentine surface is important for the function of OPTIFIL BOND. The etched enamel bevel should have a chalky white appearance.

Etching precaution:

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

5. Application of OPTIFIL BOND

Apply OPTIFIL BOND generously with a brush onto the enamel and dentin surfaces

for 30 seconds with agitation. The material should build a homogeneous layer. Remove excess material carefully. Dry cautiously with oil free air for about 15 seconds to remove all volatile components and to disperse the adhesive to an even layer. Do not desiccate the dentine.

Cure the OPTIFIL BOND coating by exposing its entire area to a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least 1000mW/cm² for 20 seconds before placement of a composite.

Notes:

Do not rinse off the OPTIFIL BOND! If not used immediately, place dispensed OPTIFIL BOND in subdued light to prevent premature polymerization by incident light.

To test the polymerization behavior of the material place 1-2 drops in a mixing well and light cure for 20 seconds. After that the material is hard.

6. Restorative Placement

Apply the restorative material according to the instructions of the manufacturer.

Best results are obtained with application of a thin layer of a light cure flowable composite (e.g. OPTIFILFLO) followed by the application of a moldable composite (e.g. OPTIFIL). Light cure each composite layer separately according to the corresponding user instructions.

Storage of OPTIFIL BOND

Do not store above 25 °C (77 °F)!

Avoid storage in direct sunlight.

Do not use after expiration date.

Additional Notes/Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.

- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Composition

Hydrophilic methacrylates, MDP, ethanol, photo initiators.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

SanaPro Dental GmbH warrants this product will be free from defects in material and manufacture. SanaPro Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and SanaPro Dental GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, SanaPro Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Distribuito in esclusiva da:
IDS SpA - Via San Cristoforo 28/10 - 17100 Savona (SV) - ITALY
Tel +39 019 862080 - www.ids dental.it - email: info@idsdental.it

Manufacturer:
SanaPro Dental GmbH
Stresemannstrasse 46
27570 Bremerhaven - Germany

Item 900324