

ULTRA⁺

BlancOne® ULTRA⁺ clinical cases



Medical treatment

Protocol: 3 cycles of 8 minutes



After a long trip in Columbia,
where the patient has smoked and drank,
he wanted to regain back a white smile.

Black & White photo allow
to evaluate remineralizing action
and a better view of shades improvement


Initial color: 2 M2
Final color: 0.5 M1
10 shades improvement

Medical treatment

Protocol: 3 cycles of 10 minutes



The patient requires a revision
of the aesthetic reconstruction of tooth 1.1.

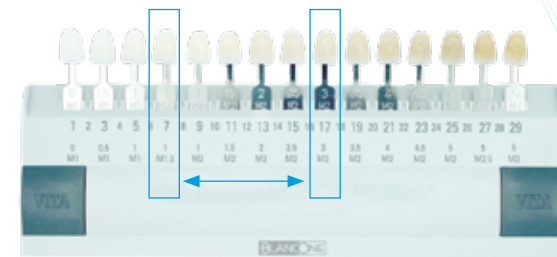

Initial color: 3 M2

Final color: 1 M1.5

10 shades improvement

Black & White photo allow to evaluate
remineralizing action and a better view
of shades improvement

Treatment of a non-vital tooth, in-clinic technique:

cleaning of the pulp chamber, sealing of the root canal with glass ionomer cement.

Application of Ultra⁺ inside and outside the tooth.

The gel was applied 3-5 times a week.



BlancOne® ULTRA+ e BlancOne® HOME+ NIGHT

Dr. Giuseppe Lipani

Treatment of a non-vital tooth, mixed technique

The gel was applied to the vestibular surface of the root canal using the black tip of the rubber dam; the gel was light-activated using a curing light for 60 seconds and left to act for 3 minutes; the procedure was repeated at least three times. Subsequently, four applications were carried out using the Walking Bleach technique.



HP35%

ULTRA+

CP12%

HOME+
NIGHT

